


uTV 3 quick start guide

Your uTV™ 3 at a glance

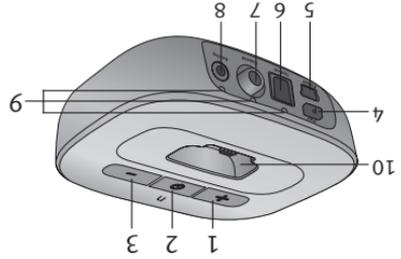

- 1 Volume up (+) button
- 2 Power on/off button and main indicator light
- 3 Volume down (-) button
- 4 Pairing (P) button
- 5 Power supply
- 6 Digital audio (optical)
- 7 Digital audio (coaxial)
- 8 Analog audio (3.5 mm)
- 9 Audio indicator lights
- 10 Exchangable charging slot

Understanding the
indicator lights

| | | |
|-----------------------|---------------------|-----------|
| Switched on | Solid green | |
| On but no audio input | Short blink green | • • • • • |
| Pairing mode | Rapid blue blinking | • • • • • |
| Streaming mode | Solid blue | ————— |
| Pause mode | Short blink blue | • • • • • |
| Test mode | Multicolor blink | • • • • • |



Unitron Hearing
20 Beasley Drive, P.O. Box 9017,
Kitchener, ON N2G 4X1 Canada



Unitron Hearing GmbH
Max-Eyth-Straße 20, 70736
Fellbach-Oeffingen, Germany

Distributed by:
Hansaton Akustik GmbH
Sachsenkamp 5
D-20097 Hamburg, Germany



Step 1



Step 2

Step 3

uDirect 2/3



2 seconds

uStream




2



Step 4


Step 5

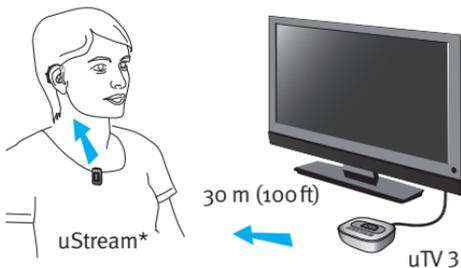

30 m (100 ft)


uStream*

uTV 3

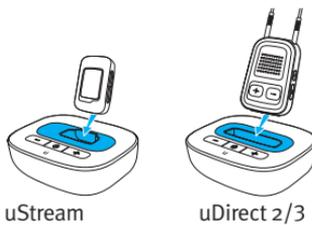

uStream

uDirect 2/3

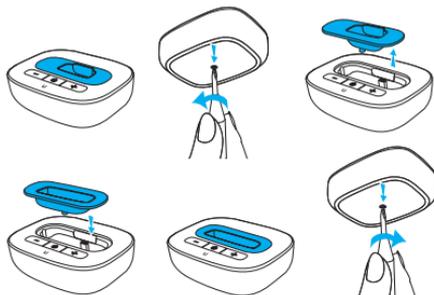

*uDirect 2/3 is also compatible.