



Hearing with ease.

Wireless CROS* or BiCROS using wireless streaming


Hearing with ease.

Use this guide to program a pair of ease products with BiPhone functionality as a wireless CROS* or BiCROS using the BiPhone program.

In this example, the Right ear is the unaidable ear and the Left ear is the better ear with some hearing loss.

Step 1

Detect the Hearing Instruments

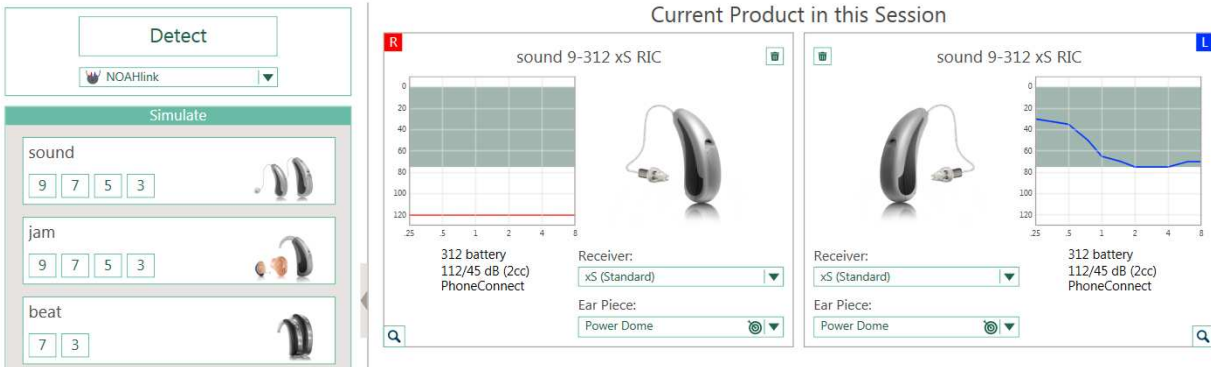


Make selections on the Acoustics & Accessories screens as appropriate. Run the Feedback manager as usual, but it is only necessary to tune it for the better (in this case Left) ear.

Step 2

Fitting / Program Manager screen

- Add a Telephone or an Acoustic Telephone program:

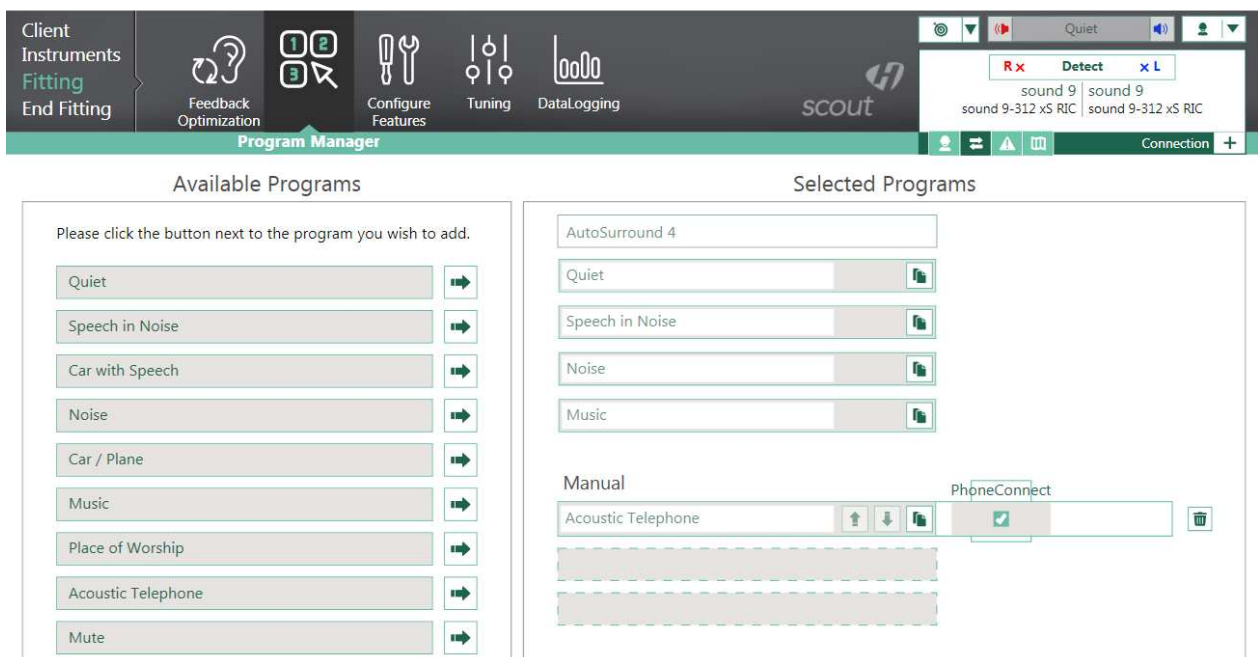

- Change program name to CROS or BiCROS if desired
- Uncheck the PhoneConnect box

Manual




*Technically, since the mic on the better side cannot be turned off in the software, these aids cannot be programmed as a true CROS fitting. Gain for a normal hearing ear will be very low and therefore unnoticeable.

Step 3


Fitting / Configure Features screen

- From the menu on the left select the CROS or BiCros program
- Enable BiPhone
- Preferred Phone Ear = unaidable ear or transmitter

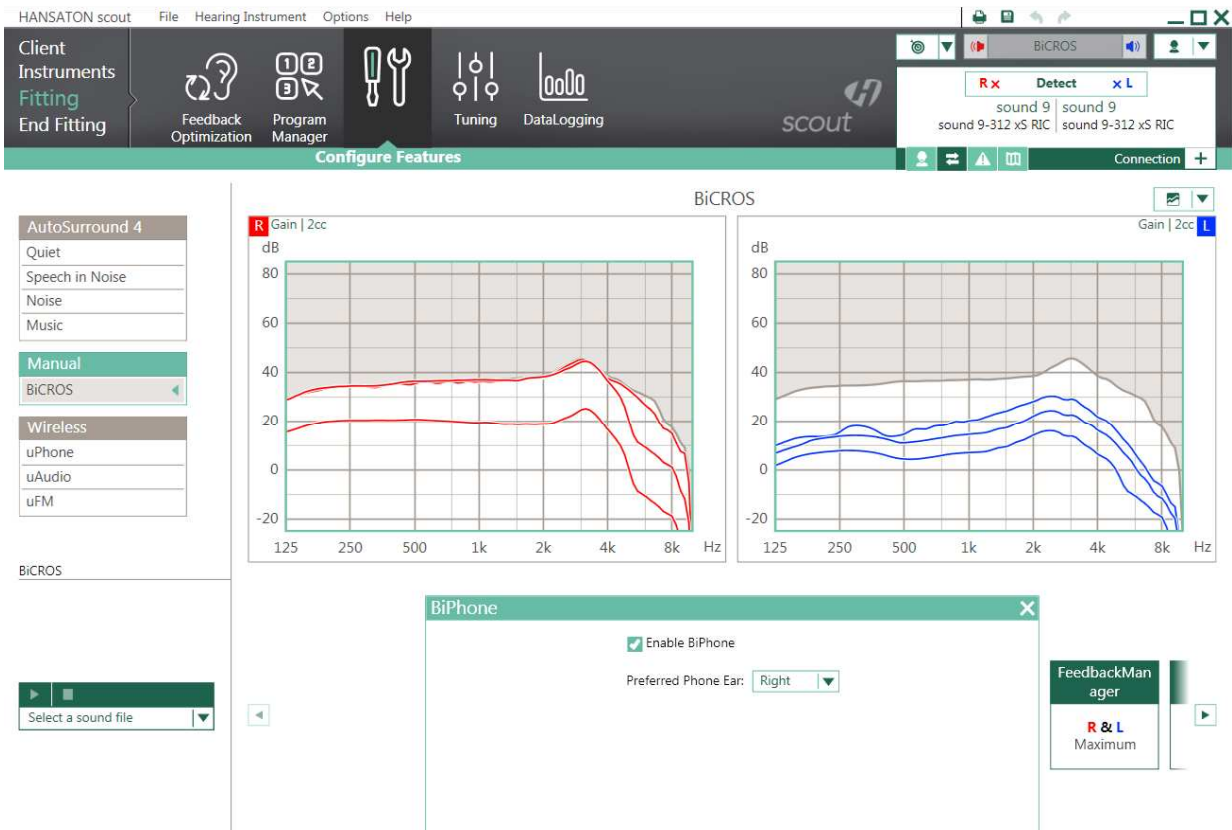

In the Input box


- Input on the Phone Ear should be "Omni (Mic)"
- Input on the Non-Phone Ear (= Better Hearing side) should be "Audio Streaming + Mic"
- Uncheck "6 dB Mic Attenuation"

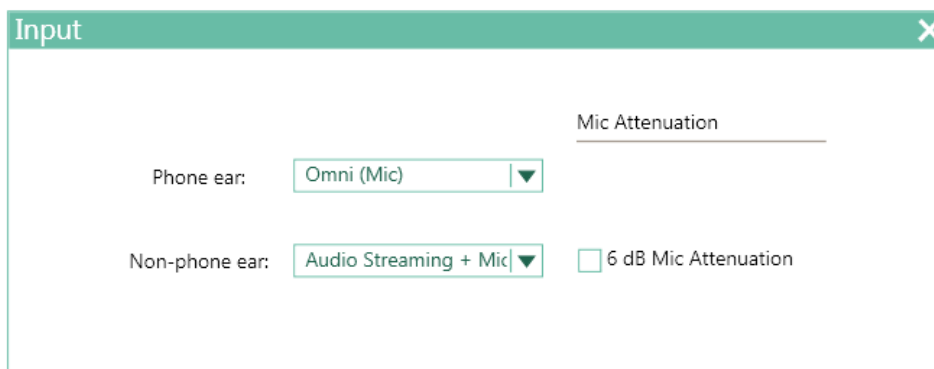


Step 4

Fitting / Tuning screen

- Select the CROS or BiCROS program from the menu on the left
- Click on the AcclimatizationManager Box at the bottom of the screen
- Click the "Unlink" button followed by unchecking "Automatic" on the unaidable side.
 - Next, slide the green "Current" slider to 100% on that side

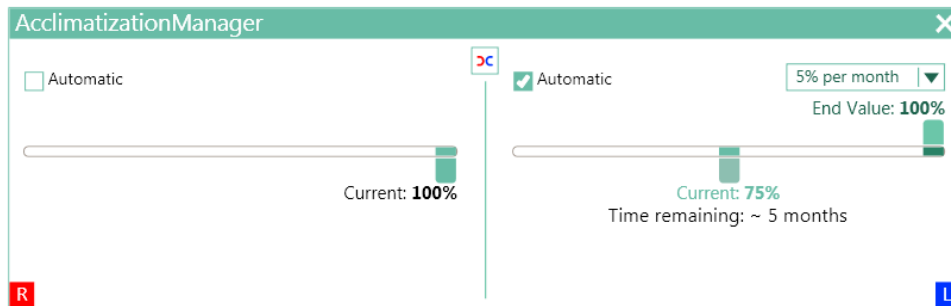

- Click on the Soft/Mod/Loud Controls box at the bottom of the screen
- Click the "Unlink" button, followed by clicking on the word "All" for the unaidable side
- Reduce gain completely in all bands. This will stop sound from exiting the earhook of the hearing instrument on the unaidable ear and stream sound (the input to the mic on that side) only to the better ear

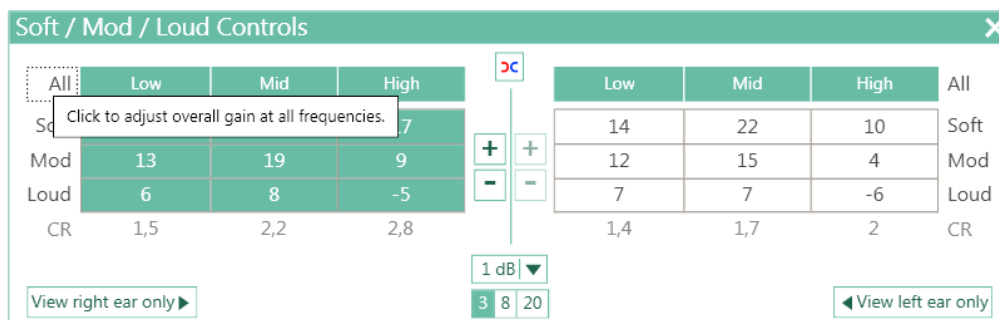

Hearing with ease.

BiCROS Tuning

Gain | 2cc

dB

125 250 500 1k 2k 4k 8k Hz

Gain | 2cc

dB

125 250 500 1k 2k 4k 8k Hz

	Low	Mid	High
All			
Soft	-16	-19	-30
Mod	-20	-28	-38
Loud	-27	-39	-50
CR	1,5	2,2	2,8

View right ear only | View left ear only

Step 5 End Fitting / HI Setup screen

- Click on the Startup Settings box at the bottom of the screen and change the Startup program to program 2 (in this example the BiCROS program)

HI Setup

Hearing Instrument Setup

Startup Settings

Startup Program: BiCROS

Startup Delay: BiCROS

Volume Settings: R & L 30dB

Data Logging: On (Logging)

Program Toggle: AutoSurround 4 / PhoneConnect

Hearing with ease.

- The hearing aid will now boot up in the second program
- Click on the Program Toggle box and uncheck "Include AutoSurround 2/3/4" which will make BiCROS the only program on this aid
- More than one Telephone or Acoustic Telephone program can be added and configured similarly if needed



- Now the BiPhone program is configured as a CROS* or BiCROS, transmitting the sound from the unaidable ear to the better ear via wireless audio streaming
- Save to the aid & database (and accessories, if any)

Remember that while connected to the computer for programming, the sound will not stream to the better ear. You need to reboot the hearing instruments (remove any cables or iCube II and restart the hearing aids) to have your patient experience this wireless CROS* or BiCROS solution

For additional information regarding HANSATON as well as HANSATON hearing systems and services, please contact us personally or on the Internet at www.hansaton.com



Hansaton Akustik GmbH • Sachsenkamp 5 • 20097 Hamburg • Germany
Telephone: +49 40 29 80 11 0 • Fax: +49 40 29 80 11 860
E-mail: info@hansaton.de • Internet: www.hansaton.com